

Askina® Heel

Anatomically shaped hydrocellular heel dressing

How to apply the Askina® Heel



1. Apply self adhesive labels on each side of Askina® Heel.



2. Apply gently the dressing on the heel area and fix with the strap.



3. Correctly positioned Askina® Heel: the ankle is covered.



4. The dressing can be also fixed with a compression stocking.

TIP:

1. The strap can be opened and closed many times to ensure that the dressing is securely and comfortably held in place.
2. Wound can be easily inspected, without patient discomfort.
3. Ensure foam heel has the triangles facing upwards for positioning on the heel.