



# ASKINA® BARRIER CREAM

PROTECTANT AND MOISTURE BARRIER

B. Braun Wound Care. Right. From the start.

# Askina® Barrier Cream

## Protectant and Moisture Barrier




Effective prevention of skin breakdown associated with incontinence

Highly concentrated:  
Small amounts will cover a large surface

Does not reduce the absorbency of continence aids

Does not reduce the adherence of fixation tapes and dressings

Temporarily resists wash-off;  
no need for frequent application

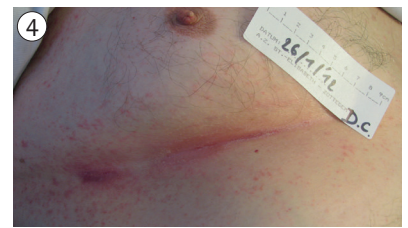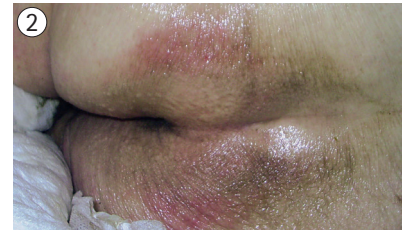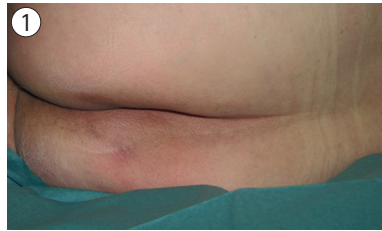

**Askina® Barrier Cream** acts as a lasting protective barrier on the skin of people living with incontinence. When applied, it is quickly absorbed, providing an invisible, breathable, moisturising and water repellent layer, resistant to washing off. It efficiently moisturises very dry skin. It can also be used for skin protection whenever there is a risk of skin breakdown due to prolonged contact with body fluids.

### INDICATIONS

Askina® Barrier Cream is indicated for use on intact skin. Askina® Barrier Cream acts as a protectant and moisture barrier:

- Against maceration caused by incontinence or body fluids
- To protect sensitive, fragile and severely dry skin, including periwound areas
- To prevent skin irritation

**Askina® Barrier Cream** has a large area of application.


**Examples:**

1. To protect the skin from incontinence related issues
2. To moisturise and protect dry and fragile periwound skin from adhesive damage and moisture exposure
3. Provides protection from all moisture related issues

### TIP

Dry the skin thoroughly before application

- Spread Askina® Barrier Cream thinly to cover the affected area
- Re-application time will be dependant on the frequency of cleaning and with secondary dressing change
- On frequently cleaned skin, daily reapplication may be required
- Can be used underneath Askina® Barrier Film for when moisture plus additional protection is required

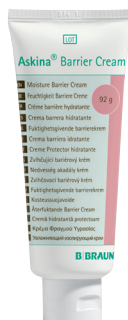

## B. Braun Ordering Information

Askina® Barrier Cream

Size	Pcs/Pack	Article number
92 g	1	4119201