

# Alleviate your day-to-day...

Millions of women around the world are suffering from stress urinary incontinence. In France alone, this affects more than 3 million women. Have you been embarrassed to laugh, go to a party or to play sports? Urine loss can affect your daily activities. It can prevent you from doing the things you love.<sup>(2)</sup>

Don't let it overwhelm you, choose Diveen®, the discreet protection!



## What is Diveen® ?

Diveen® is an intravaginal device indicated for women with urine loss, especially from stress or mixed urinary incontinence.

As simple as a tampon to apply!

Clinically proven efficacy in the management of stress urinary incontinence in women.<sup>(3)</sup>

- ☀️ Diveen® reduces frequency of urinary leakages by 69% in average <sup>(3)</sup>
- ☀️ Diveen® improves significantly patient satisfaction about urinary symptoms: satisfaction score of 78/100 for patients who used Diveen\*
- ☀️ Diveen® improves users' emotional well-being <sup>(3)</sup>


\* vs. satisfaction score of 5/100 for patients who did not use Diveen <sup>(3)</sup>

(1) Brevet EP 1 924 218 B1  
(2) Haab F. et al. Traitement de l'incontinence urinaire d'effort par colposuspension percutanée: une technique non satisfaisante. Progrès en Urologie 2001, 11:336-339  
(3) Cornu J.N et al. 75NC007 device for noninvasive stress urinary incontinence management in women: a randomized control trial. International Urogynecology Journal 2012, 23 (12): 1727-34

Note: Conflicts of interest Professor Stephane Mouly received consultancy honoraria from B. Braun, Pfizer, Sanofi-Aventis, Lilly Research Laboratories, Lundbeck and Chugai Pharma. Other authors have no conflicts of interest to declare.

# Why Diveen®?

Discreet, easy-to-use and efficient<sup>(3)</sup>, Diveen® follows you wherever you go

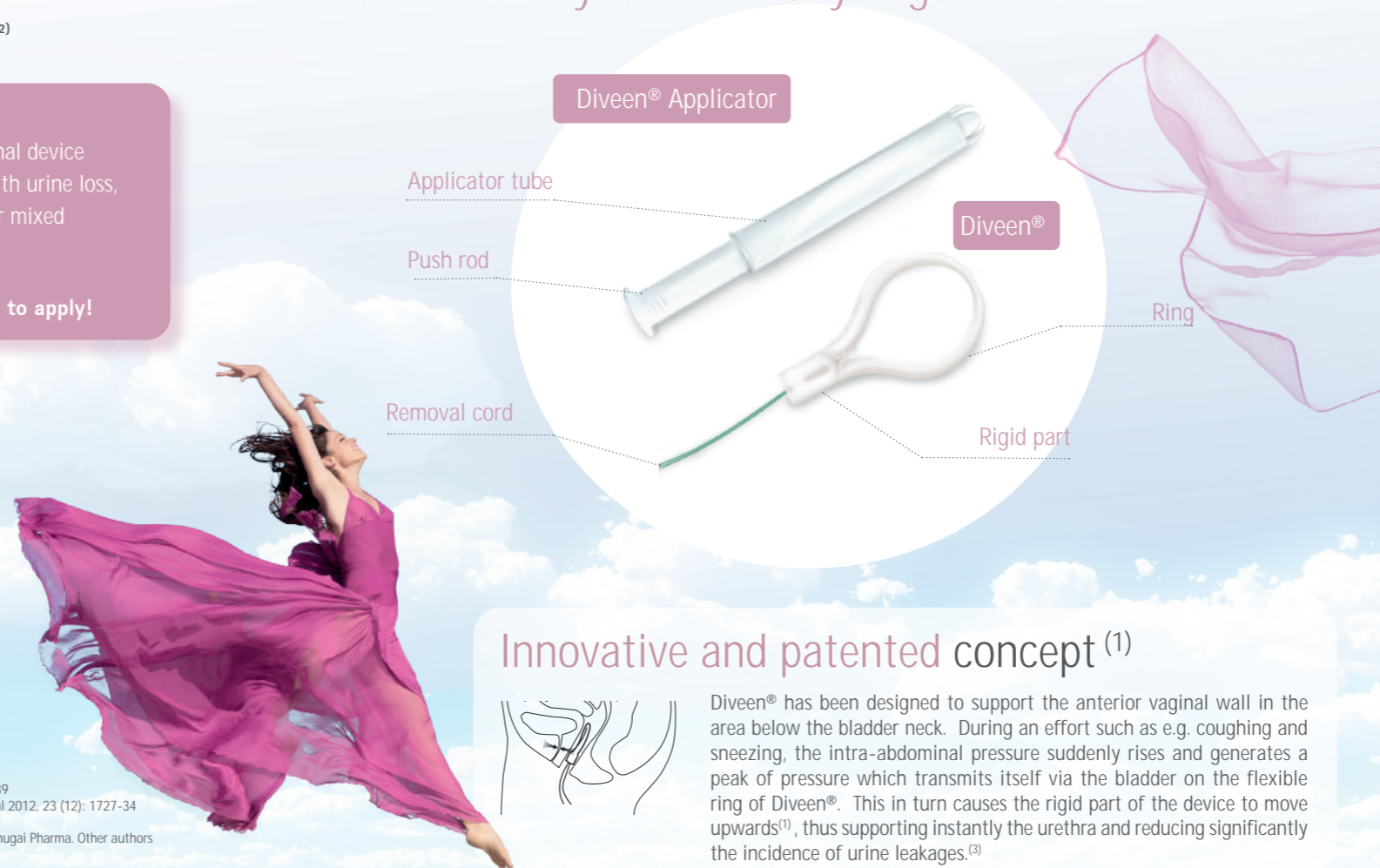

Diveen® Applicator

Applicator tube

Push rod

Removal cord

Diveen®

Ring

Rigid part

## Innovative and patented concept <sup>(1)</sup>



Diveen® has been designed to support the anterior vaginal wall in the area below the bladder neck. During an effort such as e.g. coughing and sneezing, the intra-abdominal pressure suddenly rises and generates a peak of pressure which transmits itself via the bladder on the flexible ring of Diveen®. This in turn causes the rigid part of the device to move upwards<sup>(1)</sup>, thus supporting instantly the urethra and reducing significantly the incidence of urine leakages.<sup>(3)</sup>

## Why to use Diveen® ?

- A unique mechanism enabling significant reduction of the risk of urine leakages. <sup>(1)</sup>
- 100% discreet protection.
- Biocompatible material.
- 12 hours of incontinence control to enjoy physical activity with total peace of mind. Remove the device at night after one day of use
- Easy to use, similar to a tampon. No need to remove Diveen® during your day, even when going to the toilet.
- Can be inserted without using the applicator.



<https://www.bbraun.com.au/en/products/b1/diveen.html>




# How to wear Diveen® in a few simple steps\*

## 1. Before using the device

- Wash your hands.
- Get into a comfortable position.

## 2. Preparation of the device

- Gently pinch the ring of the device & insert it into the applicator tube.
- Insert the push rod back into the applicator tube. Diveen® is now ready for insertion.

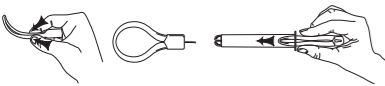

## 3. Insertion of the device

- Insert the applicator tube into the vagina.
- Using the push rod, push the device into the vagina (just like a tampon applicator)




## 4. Removal of the device

- Gently pull on the removal cord.



\* Please read the product instructions carefully before use. Consult your doctor for advice on managing your incontinence.

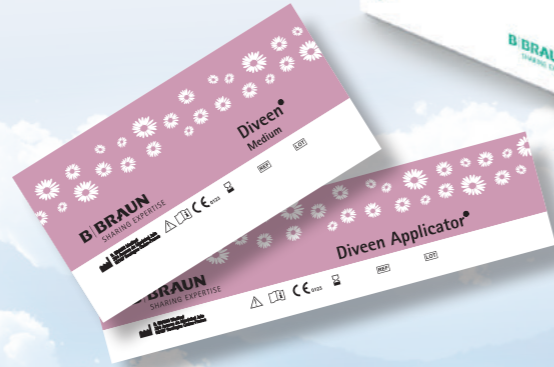

Ref	Size	Units/box
3132	Small	5 Devices +1 Applicator
3133	Medium	5 Devices +1 Applicator
3022	Small	15 Devices +1 Applicator
3023	Medium	15 Devices +1 Applicator
3025	Small + Medium Set	2 devices + 1 Applicator

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non binding documents and photographs. For healthcare professional use only. Please read carefully the instructions for use.



CE 0123

B. Braun Australia Pty Ltd | Level 5, 7-9 Irvine Place, Bella Vista NSW 2153 Australia | Tel.1800 251 705 | info@bbraun.com | www.bbbraun.com.au  
 B. Braun New Zealand | PO Box 37353, Parnell, Auckland 1052, New Zealand | Customer Care 0800 227 286 | Fax (09) 373 5601 | www.bbbraun.co.nz  
 BAUS OM E0820 10/21 | 2021 © B. Braun Australia Pty Ltd



Innovation

Feel free again!

**DIVEEN®**

The 1<sup>st</sup> intravaginal tampon designed to reduce stress urinary incontinence.<sup>(1)</sup>